
- PL** Schemat zakładania ortozy:
- EN** Fitting:
- DE** Benutzung:
- CZ** Nasazení:

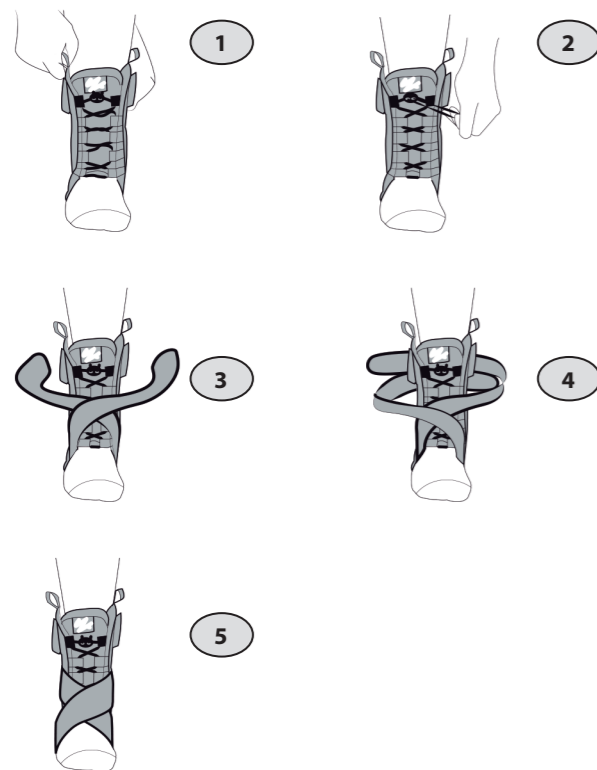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EN

**Indications:**

Treatment of ankle joint injuries: sprains I° and II°, ligaments damage. Stabilization in case of permanent or temporary instability of the ankle joint. Protection against excessive eversion and inversion movement. Habitual sprain of the ankle joint. Prevention of injuries and overloading.

**Contraindications:**

Unstable fractures require stiffening in a cast. The product is designed to be worn on a cotton sock. In case of any problems related to use of the product, please contact your doctor. Patients with impaired blood supply to the extremities, eg. with diabetes or peripheral vascular disease, must preserve extreme caution. Do not wear the brace on open wounds.

**CAUTION!**

The final decision concerning the usage and selection of the appliance shall be made by the doctor or medical specialist.

**Selection and fitting:**

1. In order to properly choose the size of the brace, measure the circumference of the foot in a sock by the calcaneus and instep. Wrong choice of the size may reduce the effectiveness of the orthosis.
2. Insert foot into the brace so that the heel is settled in designed hole. Place the tongue comfortable so dorsum of the foot is not oppress and ankle joint is protected from the lacing.
3. Pull the shoelace off by using the self-tightening clip.
4. Lead stabilizing strap from the medial side after the top of the foot, then wrap around the ankle and fasten above the medial malleolus. Lead stabilizing strap from the lateral side analogously. The force of stabilization can be adjusted by proper tension of the tapes.

5. Try to charge the limb and check whether the force of the lacing provides adequate stability. If necessary lace orthosis once again to achieve maximum comfort and proper stabilization force.

**Maintenance:**

**Before washing, remove the liners made of stainless steel. Laces should be washed separately (manually).** Orthosis can be washed in cold temp. (30°) with a mild detergent, without using liquid fabric softeners. Do not use bleach and strong detergents. In order to dry the orthosis place it on a towel and spread out and dry in the air. Do not twist, do not wring out brace. Do not tumble dry automatic or on a heater. Check the condition of orthotic after every wash.

**CAUTION!**

Failure to persevere good personal hygiene, usage of the wrong detergent can cause abrasions or other ailments associated with increased sweating and bacterial growth.

**Size chart:**

Size	Circumference of the foot by calcaneus and instep (cm)
1	20-23
2	23-25
3	25-27

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