Wydanie: 17.11.2021
Edition: 17.11.2021
Ausgabe: 17.11.2021
Vydání 17.11.2021

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EN

AFO-SOFT is an open type orthosis of wide therapeutic use. It helps with treatment and rehabilitation of ankle joint injuries and dysfunctions. Its construction supports the lateral ankle ligaments and medial orientation of the foot.

Indications:

Support in chronic ankle joint instability, treatment of injury of lateral ankle ligaments.

Follow-up treatment of ankle fracture, follow-up treatment after surgery. Recommended in case of joint capsule injury, ligament inflammation.

Product description:

AFO-SOFT is made of breathable Airx fabric and it is equipped with special “comfort areas” which are placed on vulnerable spots to protect against abrasions and to provide better stabilization. Afo-Soft is fastened with Velcro bands. There are also 8 flexible figured stabilizing straps which allow to regulate the compression strength, maintain bending stress and support collateral ligaments. Available in one, universal size.

Putting on:

Unfasten the orthosis and put it onto the foot. Fasten the Velcro straps and adjust compression to the individual needs. Fasten the stabilizing straps.

Maintenance:

The brace should be washed with gentle detergents in max. temperature 35°.

Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial heat source. Dry in horizontal position. Do not iron.

Warning!

Not observing the rules of personal hygiene and cleaning the vest with inappropriate detergents may result in skin abrasion or other effects related to perspiration. Please, do not use gels or warming ointment. Do not forget to rinse carefully after washing-detergent residue, otherwise that residue can cause skin irritation and damage product material.

Warning!

The product is not fireproof. Do not use close to the open sources of fire.

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