**Objaśnienie zastosowanych symboli:  
Description of symbols used:  
Erläuterung der verwendeten Symbole:  
Popis použitých symbolů:**

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EN

AFO-WALKER ankle-foot orthosis is a product of wide range of use. It effectively supports treatment and rehabilitation in case of injuries and dysfunctions of ankle joint and foot. It provides stabilization and allows to adjust flexion angle of plantar and dorsal flexion. Five straps enable proper adjustment and fitting.

#### Indications:

Fracture and dislocation of the metatarsal bones, acute conditions, post-operative treatment. Rapture of the tarsal bones and calcaneus. Surgery on the joint of the big toe; soft tissue injury, rapture of the Achilles tendon; severe sprain, ankle sprain. Protection and stabilization of post-traumatic conditions of foot and ankle. Immobilization of the ankle joint in selected therapeutic range. Conservative treatment after surgery.

#### Product description:

Four hook and loop fasteners allow to adjust compression to individual needs. Reinforced construction provide optimal lateral stability. The brace is equipped with hinge which allows to adjust flexion angle (plantar and dorsal, from 0° to 70°). One, universal version for left and right leg.

#### Use:

First, adjust the hinge.

Then, loosen the straps and unbutton lining.

Insert the foot, fasten the straps and lining.

Before making changes to the brace always consult a doctor or therapist.

This product is to be used under the supervision of a medical staff.

#### Maintenance:

Please, do not use solvents, harsh detergents, sharp brushes and hard objects. Lining should be washed using mild detergents at 35°C. Do not bleach or dry clean. Squeeze gently before drying. Dry away from artificial sources of heat. Do not iron.

#### CAUTION:

Lack of personal hygiene, using harsh detergents can cause abrasions or other skin problems associated with increased sweating and bacterial flora. Do not use product on gels and warming ointments. Please note that detergent which wasn't washed out can cause skin irritation and damage the material.

THIS PRODUCT IS TO BE USED UNDER THE SUPERVISION OF A MEDICAL SPECIALIST.

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