**ATTENTION!**

The brace should be used according to the instructions given by the doctor or physiotherapist. When used for the first time, the support should be put on by the experienced health-care personnel. Range of motion should be adjusted according to instructions given by the doctor or physiotherapist. Before the first use, please, read the user's manual carefully. If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

**ATTENTION!**

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

**ATTENTION!**

Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled.

Universal size.

PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

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**Product description:**

The strap uses soft knitted fabric and silicon pad to compress the patella tendon. Regulated band allows to choose optimal circumference of the strap and proper compression.

**Indications:**

- Patellofemoral syndrome
- Inflammation of patella ligament
- Osgood-Schlatter disease
- Jumpers Knee

**Contraindications:**

- Necessary lateral stabilization of patella
- Injuries of patellar ligament
- Gaping wounds, skin abrasion and other skin changes

**Putting the orthosis on:**

1. Unfasten the strap. Put on so that the silicone pad would be directly below patella, on patellar ligament.
2. Fasten the band the patellar ligament should be compressed but the band cannot restrict circulation.
3. Put hand and forearm on the splint and fasten the Velcro belts.

**Maintenance:**

The pad can be washed up by hand in lukewarm water using mild detergents. Do not chlorinate, do not whirlpool. Wring gently before drying. Dry in a distance of artificial sources of heat. Do not iron.

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