Materials:

PP, PA, Polyester, PU Foam, Aluminium, Steel.

Size*	XS	S	M	L
Thigh circumference (cm)	33 - 39	39 - 48	48 - 57	57 - 67

*Measurement 15 cm above patella.

ATTENTION!

The brace should be used according to the instructions given by the doctor or physiotherapist. When used for the first time, the support should be put on by the experienced health-care personnel. Range of motion should be adjusted according to instructions given by the doctor or physiotherapist. Before the first use, please, read the user's manual carefully. If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

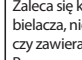
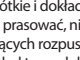
ATTENTION!

Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled.

PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

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**Product description:**

FIRST brace effectively helps in treatment and rehabilitation following injuries which require limitation of angular range of motion. Anatomically profiled, light aluminium frame. Policentric hinge provides flexion control (0°, 15°, 30°, 45°, 60°, 75°, 90°), extension control (0°, 10°, 20°, 30°, 40°) and 0° immobilization. Stabilization bands keep the brace in place and ensure effective treatment. Breathable, condyral pads placed below the hinge protect against skin abrasions and provide maximum comfort. Two extra padding elements for change. Two versions available: for right and left leg.

Indications:

- Stabilization after ACL, PCL, MCL, LCL
- Knee joint protection against recurrent injuries while returning to sport activities
- Follow-up treatment after reconstruction of ligaments in knee joint

Contraindications:

- Injuries which require complete immobilization of knee joint.

Directions for use:

1. While used for the first time, the brace should be adjusted and put on with the help of experienced health-care personnel.
2. Before using the brace for the first time, make sure that the limitation of angular range of motion has been properly chosen, according to the doctor/physiotherapists advice.
3. Always check if the setting is the same for both hinges.
4. In order to set the hinge, unscrew the bolt placed on the cover of the hinge.
5. Put the wedge with chosen extension degree.
6. Tighten the screw. Repeat the step at the back of the brace by placing the wedge with chosen flexion degree.
7. Unfasten all the stabilizing straps.
8. Put the brace so that the frame elements would be on the front part of the thigh and calf, and the lateral splints symmetrically on both sides of the knee joint. Hinge axis should be at the height of upper edge of patella.
9. Fasten the stabilizing straps following below steps in the imposed order: front thigh strap, back tibia strap, front upper tibia strap, back lower thigh strap, back upper thigh strap front lower tibia strap.
10. Set of extra pads allow to adjust brace's circumference to the knee joint.

Maintenance:

Clean the frame elements with moist cloth. Padding should be Washed by hand using gentle detergents in temperature max 35°C. Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial warmth source. Do not iron.

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