Objaśnienie zastosowanych symboli:**Description of symbols used:****Erläuterung der verwendeten Symbole:****Description des symboles utilisés:****Popis použitých symbolů:**

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**Indications:**

- Non-invasive treatment of the movement segment of different etiology and different degree of intensity.
- Stabilisation of the thoracic- lumbar- sacral segment in the cases of segmentary instability following surgeries.
- Follow-up treatment after neurological and orthopedic surgeries – functional treatment.
- Prophylaxis against recurrent injuries, occupational and sporting excessive.
- Loads Inborn faults of the thoracic- lumbar- sacral segment of the spine (with and without pain syndromes).
- Chronic pain in degenerative diseases.

Contraindications:

Conditions requiring immobilization of the orthosis affected section (lumbosacral spine section), difficult to heal wounds and other skin disorders within the area of the orthosis.

Effects:

Reduction of the bending and straightening amplitude of the lumbar spine. Balancing the pressure on the intervertebral disc and intervertebral joints. Supporting the action of weakened and relieving the over-tense muscle groups.

Use:

For hygienic and patient's comfort reasons, we recommend the lacing to be used on seamless cotton underwear. Do not forget to choose the size carefully, to ensure proper operation of the lacing.

1. Open the main Velcro of the lacing.
2. The lacing should be put on when the user is lying down, to allow stabilization of the sacro-lumbar section structures when relieved.
3. Put on the lacing so that its lower edge is about cm above the end of the coccyx (2 finger width above the cleavage), and the inner label is at the top edge.
4. The rear braces should adhere to the sacro-lumbar vicinity. Fasten the main Velcro of the lacing then fasten the tightening tapes, adjusting the pressure and level of the dorsum pressure of the corset.
5. The lacing should be fastened to ensure stabilization (partly limiting the spine movement within the sacro-lumbar section),

but not to disturb breathing.

6. The time of wearing the lacing should be strictly determined by the physician or a physiotherapist. A rehabilitation training programme should be implemented in parallel.

Maintenance:

The lacing should be hand washed with very mild washing agents at the temperature of up to 35°C, not to be treated with chlorine, not dry-washed. It should not be squeezed nor whirled dry; prior to drying press gently. Dry away from artificial sources of heat. Spread out to dry. Do not iron.

Materials:

PA, PU, PES, PCV, PE, LATEKS, METAL.

! Attention!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

! Attention!

Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled.

Size:

Size	S	M	L	XL	XXL
Waist circumference (cm)	85 – 95	95 – 105	105 – 115	115 – 125	125 – 135

PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.