ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

ATTENTION!

Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled.

PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

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EN

Product description:

AFO-FIT is a light, hygienic orthosis AFOs which supports and positions a limb with tight, contracted muscles into a more normal position. AFO-FIT is a foot-drop orthosis made of polypropylene. Due to its construction is light and leaves the calf and heel free. Self-fastening shin brace provides basic stabilisation reinforced by additional stiff brace embracing the ankle joint. Rectangular plate ensures optimal comfort of use. AFO-FIT is available in left and right leg version.

Indications:

- Prevention of foot-drop in case of peroneal nerve palsy and neurosurgical diseases.

Putting on:

1. Place the rectangular plate below the shin where the Velcro band is placed.
2. Put the orthosis into the shoe. Put on the shoe and fasten the upper band.
3. Then put the additional stiff brace above the ankle and fasten it adjusting the compression to the individual needs.

Size	S	M	L	XL
Shoe size (cm)	35 – 37	37-39	39 – 41	41 - 44

Maintenance:

Product hygiene is important for patient's rehabilitation. The brace should be washed with gentle soap and water. It can be also washed with gentle detergents. Please, do not use the strong chemical detergents and bleaches.

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