6. Fasten the chest belt (6) to the round grommet (7) so that it lies on your chest (above your breasts). Fasten the two stabilising straps (8 and 9) to the waist belt (4). Strap (8) should be fastened at the front (in the abdominal part of the waist belt) and strap (9) should be fastened at the back (in the dorsal part of the waist belt).
7. Adjust the length of all straps.
8. Fasten the shoulder and forearm stabilising straps (10, 11, 12).

REMEMBER! The straps should be fastened so that they maintain the position of the arm. The cross strap (3) should be on the opposite shoulder and not on the neck.

Maintenance:

Remove the padding of the orthosis from the frame. It can be hand washed in very mild detergents at up to 30°C; do not chlorinate, do not dry clean, do not wring, do not spin, squeeze gently before drying. Dry away from artificial heat sources. Dry unfolded. Do not iron. Put the orthosis padding back on the frame after drying.

Materials:

PU, PA, AL, ST.

Size	S	M	L
Patient's height (cm)	150 - 165	165 - 170	Above 170

ATTENTION!

Do not use the product if it has been damaged or deformed. Do not attempt to repair the product yourself. Do not use the product in a manner for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the orthosis with warming gels or ointments.



ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

ATTENTION!

Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled. PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

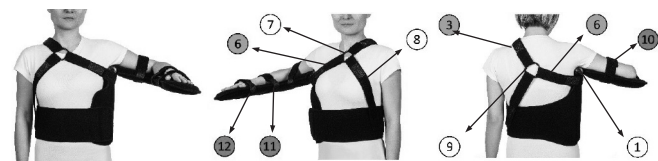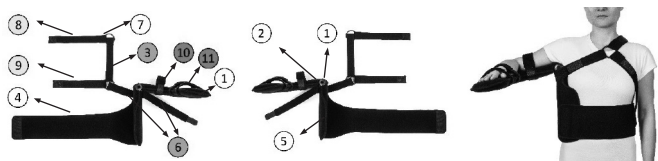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Product description:

The arm abduction orthosis provides effective stabilisation of the shoulder joint in a therapeutic position for various types of damage in this area. It can also be used as a post-surgical accessory to shorten and improve the recovery process. The design of the orthosis allows the correct angle of limb abduction to be set depending on the stage of treatment and rehabilitation and according to the recommendations of the doctor or therapist. The orthosis is made of a steel, lightweight and sturdy frame with a locking device that allows the arm to be stabilised in the appropriate therapeutic angle setting (15°, 30°, 45°, 60°, 75°, 90°). The frame is covered with a soft padding made of breathable knitted fabric for comfort. Thanks to the fully adjustable system of stabilising straps, the orthosis can be perfectly adjusted to the patient's anthropometric measurements. Properly positioned straps ensure stabilisation of the affected limb and evenly distribute its weight without overloading the opposite arm and cervical spine.

Indications:

- Treatment after shoulder dislocations
- Rotator cuff plasty or injury
- Arthroscopy of the shoulder joint
- Subcondylar fracture of the humerus
- Bankart lesion
- Post-operative stabilisation of the shoulder joint
- Endoprosthesis of the shoulder joint

Contraindications:

- Conditions requiring immobilisation in a different therapeutic position (adduction, internal rotation).
- Special care should be taken with the use of the orthosis in patients with sensory disturbances in the area of the orthosis effect and special control should be given to patients with intellectual dysfunctions.
- In case of severe pain occurring during the use of the orthosis, contact your doctor or therapist immediately.

Disposal of the medical device:

Conditions requiring immobilisation in a different therapeutic position (adduction, internal rotation).

Target groups of patients:

The orthosis is designed for patients with shoulder joint dysfunction, after injuries, diseases, to support the treatment and rehabilitation process by immobilising the arm in the abduction position according to the therapy requirements - 15°, 30°, 45°, 60°, 75°, 90°.

Putting on:

1. Adjust the orthosis to immobilise the correct limb - unscrew the knob (1) located at the connection of the frame elements and adjust the arm and forearm support element (2) in a position to immobilise the right (fig. A) or left (fig. B) arm.
2. Set the desired therapeutic angle of abduction before putting on the orthosis.
 - a) Unscrew the black knob located at the connection of the orthosis frame components (1).
 - b) Then set the desired angle and tighten the knob. The arm and forearm support element changes its position in steps. Each position marked with a white line means a change of 15° in the angle setting. The arm and forearm support element (1) is permanently positioned in the position marked by the line. The lower frame element (5) positioned on the first line means that the limb will be immobilised in a 90° position.
3. To put on the orthosis, prepare it as shown in figure A,C for the left limb or figure B, D for the right limb.
4. Place the arm on the arm and forearm support element (2) so that the connecting element (1) is positioned in the axillary fossa. Fasten the waist strap (4) around the waist with the Velcro provided so that the vertical frame element (5) fits close against the side. If necessary, move the velcro to the correct side of the waist belt to be able to fasten it.
5. Fasten the straps so that the cross strap (3) is on the opposite shoulder.

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