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EN

Product description:

The Manu medical Plus wrist orthosis is a highly effective orthosis on the latest technical level. The material is permeable to air which creates a comfortable microclimate and high wearing comfort. The three Velcro fastening combined with the anatomically preformed metal insert allow a steady immobilization and stabilization of the wrist. The immobilization of the thumb metacarpophalangeal joint is realized by means of Velcro fastening.

Safety instructions:

It is recommended that the first fitting of the Manu Medical Plus orthosis be performed by trained medical personnel. Basically, when adjusting the Velcro straps it has to be ensured that they are not fastened too tight to avoid congestion.

Employment according to purpose and indications:

The Manu medical Plus is used in case of soft-tissue irritations of the wrist and the thumb metacarpophalangeal and thumb saddle joint, for the conservative immobilization after distorsions and in case of tendovaginitis, carpal tunnel syndrome and rheumatoid arthritis. Any other employment, i.e departing from the direction, results in an expiration of the product liability.

Application of the bandage:

1. Close all three Velcro straps at the outer ends and pull the orthosis over hand and wrist.
2. Firstly tighten the upper strap at the back of the hand, then the middle strap around the wrist and then the outer strap. At last fix the thumb with a Velcro strap.
3. The orthosis should have an evenly tight fit without constrictions. If necessary, readjust the Velcro straps.

Contraindications:

Provided that the product is employed properly contraindications are not know. If the orthosis is too tight pressure symptoms may evolve. In case of the following clinical pictures the product must only be applied after agreement with the responsible physician: skin changes or inflammations in the area concerned, scarring, hypersensitivity because of heat production, varicose veins, circulatory disturbance in the hands and arms, lymphatic drainage disturbance.

Application duration:

The orthosis should mainly be worn during physical activity as the function only works in this state, but also for immobilization when resting.

Further information:

- The orthosis should only be worn under the care of a physician and for the intended clinical pictures.
- The Manu medical Plus is intended for treatment of one patient.
- Improper application excludes the product liability of the manufacturer.
- Should the symptoms aggravate a physician should be consulted immediately.
- A combination with other products has to be agreed with the physician in advance.

Storage information:

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

Cleaning information:

Washing by hand in lukewarm water with mild detergent. Please hang up the bandage dripping wet.

Guarantee:

The guarantee lies within the framework of the legal regulations.

Materials:

PA, AL, PP, PES, CR, PU

ATTENTION!

Do not use the product if it has been damaged or deformed. Do not attempt to repair the product yourself. Do not use the product in a manner for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the orthosis with warming gels or ointments.

ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

ATTENTION!

Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled.

PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

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