4. The forearm loop is passed around the circumference of the healthy side, and attached to the back side of the arm site and adjusted to the correct length.

#### Instructions for cleaning:

Padding and tightening straps should be hand-washed at low temperatures (30°) with mild detergents, without softeners. Do not use bleach or strong detergents. Do not spin, do not wring, gently squeeze before drying. To dry, spread the parts of the upholstery on a towel and air-dry. Keep away from artificial sources of heat. Check the condition of the orthosis after each wash. Clean the product systematically, at least twice a month, and more often if necessary.

#### Composition:

PES, PA, PU, PP

#### Intended user:

The medical device is intended for regular (non-professional) users.

The medical device is intended for regular (non-professional user) as well as for the professional user.

#### ATTENTION!

Do not use the device if you have circulatory disorders.

Do not use the device if it has been damaged or deformed.

Do not attempt to repair the device yourself.

Do not use the device of the wrong size or in a way for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the orthosis with warming gels or ointments. Any serious incident in connection with a device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is based.

IT IS RECOMMENDED TO CONSULT A SPECIALIST BEFORE USING THE DEVICE

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EN

AS-BASIC MEDICAL ARM AND SHOULDER ORTHOSIS

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**Intended use**

- post-operative immobilisation of the shoulder joint and arm.

**Indications:**

- treatment, recovery after injuries and traumas of the shoulder area, shoulder girdle, shoulder and elbow joints requiring immobilisation of the limb in adduction and internal rotation,
- inflammatory and degenerative conditions requiring immobilisation and pressure relief of the upper limb,
- dislocations of the acromioclavicular or sternoclavicular joint.

**Contraindications:**

- hard-to-heal wounds or skin lesions within the area of the orthosis,
- allergic reaction to the materials used.

**Target groups of patients:**

The device is intended for patients who have undergone trauma or surgery to the shoulder joint to support the healing and rehabilitation by relieving pressure on the upper limb.

**Disposal of the medical device:**

The device must be disposed of following the applicable local environmental regulations.

**Choosing your orthosis:**

1. The first time the orthosis is fitted, it should be done upon prescription and under the supervision of qualified medical personnel. The doctor or physiotherapist should decide how much stabilisation and compression the shoulder needs.
2. The orthosis should be selected based on the dimensions shown in the size chart.
3. If the device becomes too loose or changes its position, it must be refitted starting from the first step in the section, 'How to put the orthosis on'.

**Size chart:**

Universal orthosis for right and left forearm. Available in three sizes.

Size	1	2	3
Forearm length (cm)	30-36	36-42	42-48

**How to put the orthosis on:**

Optimal fitting of the brace by qualified medical personnel is essential to ensure that your shoulder is properly stabilised.

1. The shoulder strap should be loosely positioned and fastened.
2. The forearm should be placed in the appropriate position, with the thumb placed in a loop close to the body and the fingers guided through the loops away from the body.
3. The forearm loop should be placed over the head and side of the healthy arm and adjusted without tightening.

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