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EN

## Product description:

The Lumbo medical light active bandage is a highly effective medical product on the latest technical level. The anatomically knitted form offers a very good fit and high wearability. The compression of the bandage stimulates the metabolism and decreases muscle tension which brings relief and alleviation of pain. The four integrated stabilisation struts provide additional stability and support an anatomically correct posture. Safety instructions. It is recommended that the first fitting of the LUMBO medical Light orthosis be performed by trained medical personnel. Basically, when adjusting the Velcro straps it has to be ensured that they are not fastened too tight to avoid congestion.

## Employment according to purpose and indications:

The Lumbo medical Light is used in case of general backache, debility of the back muscles, lumbagos, irritations of the iliosacral joints, attrition of the spinal column. Any other employment, i.e. departing from the direction, results in an expiration of the product liability.

## Application of the bandage:

1. Lay the bandage around the lumbar area and hold the left loop of the bandage in the stomach region.
2. Now pull the right loop over the left and fix the bandage by means of the Velcro fastening.
3. The bandage should have a steady, tight fit avoiding constrictions.
4. If necessary, readjust the fit of the bandage.
5. The metal struts can be adjusted in the back area if required. In this case please contact specialist dealer or the responsible physician.
6. The exchange of the centre metal struts by plastic struts must only be carried out by authorised per-sonnel or the responsible physician.

## Contraindications:

Provided that the product is employed properly con-traindications are not know. If the bandage is too tight pressure symptoms may evolve.

## Application duration:

The success of the bandage can be directly connected with the wearing period. According to latest investigations a permanent application does not weaken the muscles.

## Further information:

- The active bandage should only be worn under the care of a physician and for the intended clinical pictures.
- The Lumbo medical light is intended for the treatment of one patient.
- Improper application excludes the product liability of the manufacturer.
- Should the symptoms aggravate a physician should be consulted immediately.
- A combination with other products has to be agreed with the physician in advance.

## Guarantee:

The guarantee lies within the framework of the legal regulations.

## Storage information:

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

## Cleaning information:

Before washing, remove the stabilizing splints. The brace should be hand-washed in low temperatures (30°) with mild detergents, without using fabric softeners. Do not use bleach or harsh detergents. Do not spin-dry or wring; gently press out excess water before drying. To dry, lay the brace flat on a towel and air-dry. Keep away from artificial heat sources. Check the condition of the brace after each wash. Clean the product regularly, at least twice a month, or more often if needed.

## Materials:

PA, PES, PU, PP, ST

## Intended user:

The medical device is intended for both non-professionals (lay users) and professional users.

## ! ATTENTION!

Do not use the product if it has been damaged or deformed. Do not attempt to repair the product yourself. Do not use the product in a manner for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor.

## ! ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

## ! ATTENTION!

Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled.

PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

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