Objasnienie zastosowanych symboli:
Description of symbols used:
Erläuterung der verwendeten Symbole:
Popis použitých symbolů:

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**Indications:**

- Multi-level discopathy.
- Spine instability.
- Degenerative changes.
- Pain syndrome of differen aetiology.
- Post-operative stabilization.
- Osteoporotic changes.

Contraindications:

Conditions requiring immobilization of the orthosis affected section, difficult to heal wounds and other skin disorders within the area of the orthosis. Special attention should be paid to patients with abnormal heart rhythm, angina pectoris, gastric ulcers.

Effects:

Reduction of the bending and straightening amplitude of the lumbar spine. Balancing the pressure on the intervertebral disc and intervertebral joints. Supporting the action of weakened and relieving the over-tense muscle groups. Mechanical stabilization

Use:

For hygienic and patient's comfort reasons we recommend the lacing be used on seamless cotton underwear. Do not forget to choose he size carefully, to ensure properoperation of the lacing.

1. Open the main Velcro of the lacing.
2. The lacing should be put on when the user is lying down, to allow stabilization of the sacro-lumbar section structures when relieved.
3. Put on the lacing so that its lower edge is about 2 cm above the end of the coccyx (2 finger width above the cleavage), and the inner label is at the top edge.
4. The rear braces should adhere to the sacro-lumbar vicinity.
5. Fasten the main Velcro of the lacing then fasten the tightening tapes, adjusting the pressure and level of the dorsum pressure of the corset.
6. The lacing should be fastened to ensure stabilization (partly limiting the spine movement within the sacro-lumbar section), but not to disturb breathing.
7. The time of wearing the lacing should be strictly determined by the physician or a physiotherapist. A rehabilitation training programme should be implemented in parallel.

Maintenance:

The belt should be hand washed with very mild washing agents at the temperature of up to 35°C, not to be treated with chlorine, not dry-washed. It should not be squeezed nor whirled dry; prior to drying press gently. Dry away from

artificial sources of heat. Spread out to dry. Do not iron. The product can be washed in a washing machine - hand wash program.

Materials:

PA, PU, PES, PCV, PE, LATEKS, METAL

Intended user:

The medical device is intended for both non-professionals (lay users) and professional users.

! Attention!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

! Attention!

Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled.

Size:

Size	S	M	L	XL	XXL
Waist circumference (cm)	85 – 95	95 – 105	105 – 115	115 – 125	125 – 135

PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

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