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### EN

### Use:

KID Air Walking Boot provides stabilization and protection of foot and ankle joint area. Follow-up treatment in case of stable fractures of ankle/instep.

### Contraindications:

- The air cells must not be overinflated, as it may lead to skin irritation.
- Special attention should be paid in case of diabetic patients and patients with sensory disturbances.
- In case of skin changes or irritations, the compression should be decreased.
- The product cannot be used in case of open, unstable fractures, to heal wounds, sores, and other skin changes.
- It is necessary to contact a doctor in case of pain or numbness.
- Special attention should be paid while walking on wet or slippery surfaces.
- It is necessary to contact a physiotherapist or a doctor before changing the way of using the brace.
- The product should be used only under careful medical supervision.

### Placement instruction:

- Unfasten straps and remove the inner soft liner from the boot.
- Open the soft liner and place the foot inside with the heel snugly into the back portion of the liner. Fasten and wrap the foot, then the leg portion of the liner. Ensure the liner is snug but not constrictive. Make sure the pump is facing outward.
- Gently spread the side struts and place the lower leg into the brace.
- The ankle should be centered at the struts with the foot and ankle at a 90° angle.
- Fasten the Velcro straps beginning at the toe, then moving up the leg. Do not over-tighten. As swelling patterns change throughout the day\*, it may be necessary to adjust the straps for comfort. (\*Due to activities, rest, etc.)
- Remove the protective inserts placed on the shoe supports by pulling on the blue ends labeled PULL UP.
- To inflate the air cells, press the pump button located at the front of the orthosis until the desired compression is achieved.
- To deflate, turn the valve at the front of the orthosis to release air.
- To remove, unfasten the straps without removing them completely from the walker’s frame. Unfasten the soft liner and gently slide the foot and ankle out of the walker.

### Washing intructions:

Remove foam liner for hand wash, using a mild detergent and cold water, air dry. Do not wash in washing machine or in mechanical dryer. For plastic components, clean with a damp cloth, air dry. Do not use detergent or bleach.

### Maintenance:

The inner liner should be cleaned using a sponge and mild detergent (but is should not soak through). Dry far away from artificial source of heat. Dry in a flat position. Do not iron. The cover should be cleaned with a moist cloth.

### Materials:

PP, TPR, PU, EVA, PA, St, TPU

### Intended user:

The medical device is intended for both non-professionals (lay users) and professional users.

### CAUTION:

- If discomfort or pain results, persists or increases, discontinue to use and consult with medical professional.
- Make sure the hook and loop fasteners are well fastened while wearing the product. If the hook and loop fasteners can not be Fastened well, stop using the product to prevent injuries.
- It is recommended to inflate the air at least once every 8 hours during the use of the product.

### ATTENTION!

Do not use the product if it has been damaged or deformed. Do not attempt to repair the product yourself. Do not use the product in a manner for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor.

### ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

### ATTENTION!

Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled. Please seek medical advice prior to using this product.

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