Objaśnienie zastosowanych symboli:
Description of symbols used:
Erläuterung der verwendeten Symbole:
Popis použitých symbolů:

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EN

! ATTENTION:

The brace should be used according to the instructions given by the doctor or physiotherapist.

When used for the first time, the support should be put on by the experienced health-care personnel. Before the first use, please, read the user's manual carefully.

If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

Use:

Clavicle support of wide therapeutic use. It effectively helps rehabilitation and treatment in case of collar bone injuries and facilitates correction of postural defects. It can be also used as a preventive aid to establish the habit of correct body posture.

Indications:

Collar bone injuries which do not require surgical and orthopedic intervention.

Postural defects which do not require surgical intervention.

Pain syndromes of thoracic spine.

Thoracic hyperkyphosis.

Contraindications:

Skin abrasions, recent wounds and other skin problems.

Product description:

The support consists of bands made of knitted fabric laminated with foam. Special construction and wide range of adjustment possibilities allow to correct the body posture in patients at different ages.

Application:

The support consists of bands made of knitted fabric laminated with foam. Special construction and wide range of adjustment possibilities allow to correct the body posture in patients at different ages.

Size: universal

Maintenance:

The product should be washed up by hand using gentle detergents in lukewarm water. Do not chlorinate, do not dry clean, do not whirlpool, wring gently before drying. Do not dry close to the artificial warmth source. Dry in horizontal position. Do not iron.

Materials:

PA, PU, PES, POM

Intended user:

The medical device is intended for both non-professionals (lay users) and professional users.

! ATTENTION!

Do not use the product if it has been damaged or deformed. Do not attempt to repair the product yourself. Do not use the product in a manner for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor.

! ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

! ATTENTION!

Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled.

Please seek medical advice prior to using this product.

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