DE

Objaśnienie zastosowanych symboli:  
Description of symbols used:  
Erläuterung der verwendeten Symbole:  
Popis použitých symbolů:

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EN

**Indications:**

Treatment of wrist and thumb injuries and distal forearm, chronic wrist and thumb instability with pain syndrome, convalescence after surgical or conservative treatment, wrist and thumb protection in sports or occupational risk professions (overloads).

**CAUTION!**

The brace should be used according to the instructions given by the doctor or physiotherapist.

When used for the first time, the support should be put on by the experienced health-care personnel. Before the first use, please, read the user's manual carefully. If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

**CAUTION!**

Not observing the rules of personal hygiene and cleaning the orthosis with inappropriate detergents may result in skin abrasion or other effects related to sweating. Please, do not use the warming ointment.

**Putting on:**

1. Set the straps to most loosening setting for easier application.
2. Match the universal thumb-up splint.
3. Slide affected wrist into the brace.
4. Adjust the brace to a snug fit.

**Maintenance:**

Orthosis can be washed by hand in a very mild detergents at temperatures up to 35°C, after removing the aluminum stay. Do not chlorinate, do not dry clean, do not wring, do not spin, squeeze gently before drying. Dry away from artificial sources of heat. Dry in exploded form. Do not dry.

**Universal size**

PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

**NOTE!**

Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled.

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